



---

otter tub stand  
item # OT 8010

---

---

## recommended use

---

The Otter Tub Stand elevates the Otter Bath Chair in the bathtub. The tub stand legs are adjustable and bath chair mounts easily on stand. All adjustments are tool free.

---

## assembly

---

Your Otter Tub Stand comes in one carton completely assembled.  
If your shipment is incomplete, please call a Drive authorized dealer.

---

## maintenance and care

---

1. Inspect the tub stand periodically for cracks, breaks, loose or missing parts or malfunctions. If a product needs service or replacement parts, contact an authorized Drive dealer.
2. Wash tub stand with soap and water or 10% bleach.

---

## adjustments

---

See Figure 2

1. Mount chair onto base by pushing down firmly on chair until legs snap into sockets (A) of tub stand. To remove chair from stand, depress red levers (B) one at a time and pull chair upwards.
2. To adjust legs (C), pull crossbar (D) outward, rotate legs and push in to engage.

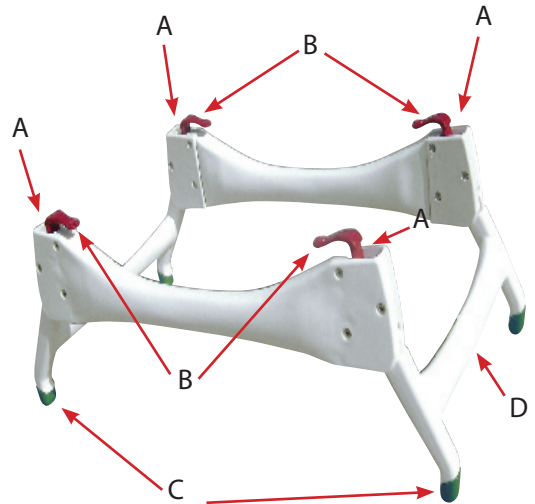


figure 2